



# **PESSARY GUIDELINES / PESARIOS**

MY MEDICAL pessaries are 100% silicone pessaries that are used in the management of vaginal prolapse of the urogenital organs.

MY MEDICAL pessaries are soft and piable, making them easy to insert and remove. Through the many sizes and shapes that are available.

#### **PESSARY GUIDE PROLAPSE COITUS WITH STRESS RETRO INCOMPETENT** 2nd-3rd Degree **MODEL** Ist-2nd Degree **CYSTOCELE RECTOCELE** INCONTINENCE DISPLACEMENT **CERVIX PESSARY IN** Σ Σ RING YES ΣMX ΣMX OVAL YES ΣMX ΣMX **SHAATZ** YES ΣMX ΣMX DISH YES $\Sigma_{MZ}^{MZ}$ $\Sigma_{ij}^{M}$ Σ **MAR-LAND** YES $\Sigma_{M}^{M}$ X ΣMX **GELLHORN** NO ΣMZ X Σ DONUT NO ΣMZ ΣMZ ΣMX NO CUBE ΣMZ Σ ΣMZ **GERHUNG** YES XX X **XX** HODGE **YES X**X ΣMZ ΣMZ **RING KNOB** YES

### TO DESINFECT PESSARIES, USE ANY OF THESE PRECEDURES:

- Autoclave at 15lbs. of pressure at 250°F or 121°C for a period of 10 minutes.
   DONUT PESSARIES CANNOT BE AUTOCLAVED.
- Boil for 15 minutes.
- Cold sterilize with CIDEX (Johnson & Johnson) or CHLOREPHENYL (Bard-Parker)





### THE RING PESSARY





Available in nine sizes, both with and without support, this is a very common pessary for a first to second degree prolapse. The ring with support can also be used on an accompanying CYSTOCELE. Insertion is eased with th folding action of the Ring.

#### SIZES AVAILABLE

Rings w/o	Support	Rings w/ Si	upport	mm
BR2.00	#I	BR2.00S	#I	50.8
BR2.25	#2	BR2.25S	#2	57.15
BR2.50	#3	BR2.50S	#3	63.5
BR2.75	#4	BR2.75S	<b>#4</b>	69.85
BR3.00	#5	BR3.00S	#5	76.2
BR3.25	#6	BR3.25S	#6	82.55
BR3.50	#7	BR3.50S	#7	88.9
BR3.75	#8	BR3.75S	#8	95.25
BR4.00	#9	BR4.00S	#9	101.6

## THE OVAL PESSARY






is designed specifically to fit a narrow vaginal vault. It is available in nine sizes, all with support, and is used for a first to second degree PROLAPSE as well as an accompanying CYSTOCELE.

### SIZES AVAILABLE

Oval w/ Supp	mm	
BOV2.00S	#I	50.8
BOV2.25S	#2	57.15
BOV2.50S	#3	63.5
BOV2.75S	#4	69.85
BOV3.00S	#5	76.2
BOV3.25S	#6	82.55
BOV3.50S	#7	88.9
BOV3.75S	#8	95.25
BOV4.00S	#9	101.6

## **SHAATZ**



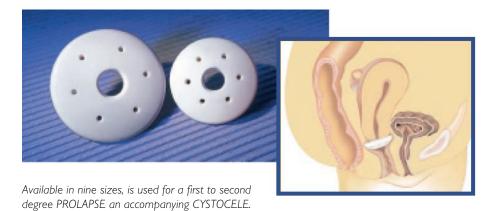

### SIZES AVAILABLE

Shaatz		mm
BSH1.50	#0	38.1
BSH1.75	#I	44.45
BSH2.00	#2	50.8
BSH2.25	#3	57.15
BSH2.50	#4	63.5
BSH2.75	#5	69.85
BSH3.00	#6	76.2
BSH3.25	#7	82.55
BSH3.50	#8	88.9





### **DISH**







The incontinense dish is used to relieve stress INCONTINENCE and minor degree of PROLAPSE. The incontinence dish comes with and without support, it is available in eight different sizes.

#### SIZES AVAILABLE

Dish w/o S	upport	Dish w/Support		mn
BDSH50	#0	BDSH50S	#0	50
BDSH55	#I	BDSH55S	#I	55
BDSH60	#2	BDSH60S	#2	60
BDSH65	#3	BDSH65S	#3	65
BDSH70	#4	BDSH70S	#4	70
BDSH75	#5	BDSH75S	#5	<b>75</b>
BDSH80	#6	BDSH80S	#6	80
BDSH85	<b>#7</b>	BDSH85S	#7	85

# MAR-LAND







Used por STRESS INCONTINENCE and minor PROLAPSE. It is available both with and without support in seven different sizes.


#### SIZES AVAILABLE

Mar-Land w/o Support		Mar-Land w/Support		mm	
	BM2.25	#2	BM2.25S	#2	57.15
	BM2.50	#3	BM2.50S	#3	63.5
	BM2.75	#4	BM2.75S	#4	69.85
	BM3.00	#5	BM3.00S	#5	76.2
	BM3.25	#6	BM3.25S	#6	82.55
	BM3.50	#7	BM3.50S	#7	88.9
	BM3.75	#8	BM3.75S	#8	95.25

## **GELLHORN**





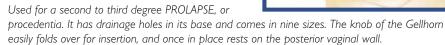

### SIZES AVAILABLE


	mm
#0	38.1
#I	44.45
#2	50.8
#3	57.15
#4	63.5
#5	69.85
#6	76.2
#7	82.55
#8	88.9
	#1 #2 #3 #4 #5 #6 #7





### DONUT



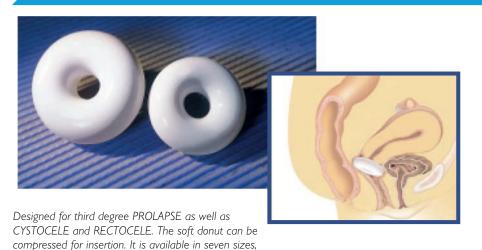


#### SIZES AVAILABLE

Donut		mm
BD2.00	#0	50.8
BD2.25	#I	57.15
BD2.50	#2	63.5
BD2.75	#3	69.85
BD3.00	#4	76.2
BD3.25	#5	82.55
BD3.50	#6	88.9

### **CUBE**



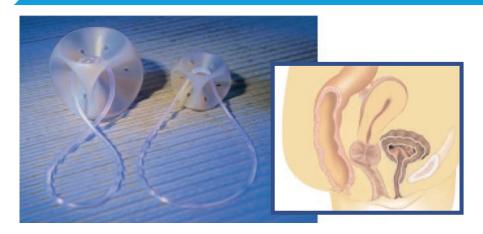


#### SIZES AVAILABLE

Cube w/o	Drain	Cube w/Dra	in	mn
BCU25	#0	BCU25D	#0	25
BCU29	#I	BCU29D	#I	29
BCU33	#2	BCU33D	#2	33
BCU37	#3	BCU37D	#3	37
BCU41	#4	BCU41D	#4	41
BCU45	#5	BCU45D	#5	45
BCU50	#6	BCU50D	#6	50
BCU56	#7	BCU56D	#7	56
BCU63	#8	BCU63D	#8	63
BCU 70	#9	BCU 70D	#9	70
BCU75	#10	BCU75D	#10	<b>75</b>

Designed for third degreePROLAPSE, including procedentia, as well as CYSTOCELE and RECTODELE. BCU75 #10 BCU75D #
The CUBE pessary is available both with and wiyhout drainage holes and has a silicone tie to aid in removal. The Cube is available in eleven sizes.

# **GEHRUNG**



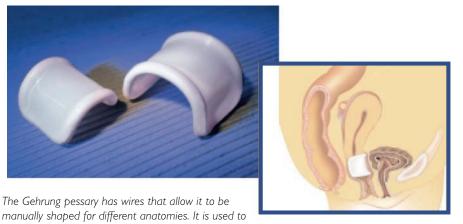

### SIZES AVAILABLE

Gehrung		mm
BGH50S	#0	50
BGH55S	#I	55
BGH60S	#2	60
BGH65S	#3	65
BGH70S	#4	70
BGH75S	#5	75
BGH80S	#6	80
BGH85S	#7	85
BGH90S	#8	90

support both CYSTOCELES and RECTOCELES as a well as second to third degree UTERINE PROLAPSE. BGT703 #6 70 It is available in nine sizes. Because of the shapable wires, the Gehrung pessary must be removed during x-rays, ultrasounds and MRIs.





### **HODGE**







Has wires that allow it to be manually shaped for different anatomies. It can be used for a first to

second degree prolapse, SYSTOCELE, STRESS INCONTINENECE and an incompetent cervix or uterine retroversion. It is available in ten sizes both with and without support. Because of the shapable wires, the Hodge pessary must be removed during x/rays, ultrasounds an MRIs.

#### SIZES AVAILABLE

Hodge w/o Support Hodge		Hodge w/ Sup	port	mm
BHD65	#0	BHD65S	#0	65
BHD70	#I	BHD70S	#I	70
BHD75	#2	BHD75S	#2	<b>75</b>
BHD80	#3	BHD80S	#3	80
BHD85	#4	BHD85S	#4	85
BHD90	#5	BHD90S	#5	90
BHD95	#6	BHD95S	#6	95
BHD100	#7	BHD100S	#7	100
BHD105	#8	BHD105S	#8	105
BHD110	#9	BHD110S	#9	110

## THE RING KNOB BOTON PESSARY







Available in nine sizes, both with and without support, this is a very common pessary for mild prolapse complicated by a mild CYSTOCELE and stress urinary incontinence.

### SIZES AVAILABLE

RKB w/o S	upport	RKB w/ Support	mm
BRK2.00	#I	BRK2.00S #I	50.8
BRK2.25	#2	BRK2.25S #2	57.15
BRK2.50	#3	BRK2.50S #3	63.5
BRK2.75	#4	BRK2.75S #4	69.85
BRK3.00	#5	BRK3.00S #5	76.2
BRK3.25	#6	BRK3.25S #6	82.55
BRK3.50	#7	BRK3.50S #7	88.9
BRK3.75	#8	BRK3.75S #8	95.25
BRK4.00	#9	BRK4.00S #9	101.6